

### Product Description

EasyPrep Mouthrinse is intended for easy intraoral isolation of patients' dentition for dental procedures. With a simple setup, the tongue, cheeks and other oral soft tissues can be retracted during dental treatment. EasyPrep Mouthpiece can be attached to dental suction units with EasyPrep Adaptor to enable efficient fluid evacuation. The EasyPrep Teeth Isolation System is designed to improve quality of dental treatment while reducing hardware and manpower requirements.

### Product Specification

Product Name: EasyPrep Mouthpiece (Non-Sterile)

Model: MGF-B16002-01 (Size M)  
MGF-B16002-02 (Size S)  
MGF-F47901 (PEDO)

Dimensions and Weight:

MGF-B16002-01 (Length / Width / High): 12.6 \* 6.2 \* 6.0 cm, 32g  
MGF-B16002-02 (Length / Width / High): 12.6 \* 6.2 \* 6.0 cm, 30g  
MGF-F47901 (Length / Width / High): 12 \* 4.4 \* 4.0 cm, 23g

Material: Medical-grade Silicone

Storage / Transportation Conditions:

5 °C to 60 °C (41 °F to 140 °F), 30% to 90% RH

Expected service life: 3 years

Listed on the ARTG: 371188



### Precautions

1. This product can be sterilized with an autoclave.
2. This product is only for dental professionals.
3. Please read this instruction manual before using the product.
4. This product is a consumable item. Please check its physical appearance before use and do not use it if there appears to be signs of aging, wrinkles or discoloration. Please contact your local retailer or distributor for replacements.
5. Please follow the cleaning guidelines when sterilizing with an autoclave.
6. This product can attach with EasyPrep Adapter to provide suction function. For purchasing information, please contact to your local retailer or distributor.
7. Please select the appropriate EasyPrep Mouthpiece size according to the dimension of the patient's mouth.

### Contraindication

- Do not use this product on patients with mandible fractures or osteoradionecrosis.
- Do not use this product on patient with facial bone fractures.

### Product Design



- Soft and flexible shield isolates tongue and cheeks, and prevents inhalation of debris.

- Multiple suction openings helps with efficient evacuation of fluid and saliva.



- Bite block designed to optimizing patient comfort.

- Transparent mouthpiece provides a clear view of airways and intraoral conditions.




### Indications

- Provides isolation and protection of oral tissue, tongue and cheek
- Increases patients' overall comfort
- Facilitates intraoral scanning and other CAD/CAM procedures
- Minimizes fogging of dental mirrors, and eliminates the need for cotton rolls
- Provides suction capabilities (requires EasyPrep Adaptor)

# Easy Prep.

Teeth Isolation System

## EasyPrep Mouthpiece Structure

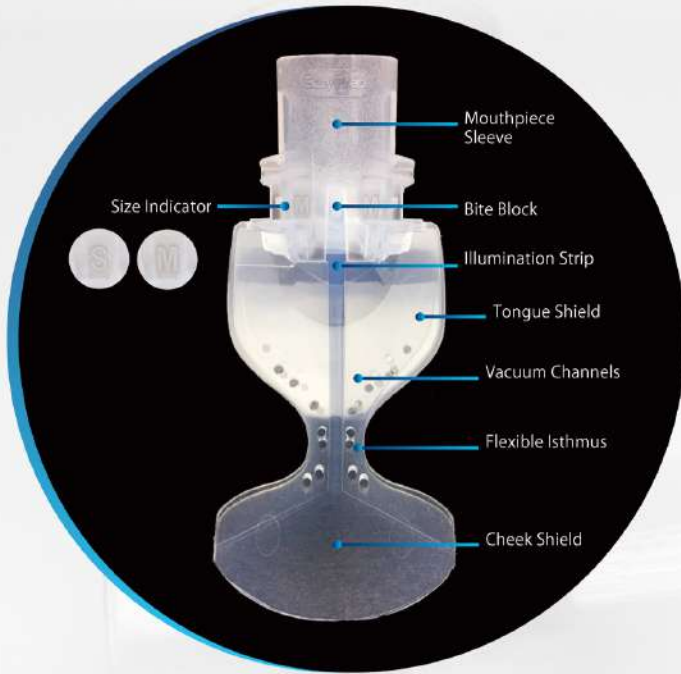

## Cleaning and Sterilization Guideline

Sterilization of EasyPrep Mouthpiece both before and after clinical use is recommended.

1. Disconnect the EasyPrep Mouthpiece from the EasyPrep Adaptor (if EasyPrep Adaptor was used with EasyPrep Mouthpiece).
2. Rinse the EasyPrep Mouthpiece and the EasyPrep Adaptor with water to clear its outer and inner surfaces off debris. Clean with medical enzymatic cleaner. **DO NOT USE COMMON CLEANING DETERGENTS.**
3. Repeat rinsing procedure until all contaminants are cleared
4. Sterilize the EasyPrep Mouthpiece and EasyPrep Adaptor by following the settings listed in Table 1.

Parameter	Autoclave Type	Temperature condition	Autoclave Time	Drying Time
1	Gravity-Displacement	121°C / 250°F	30 minutes	30 minutes
2	Gravity-Displacement	132°C / 270°F	7 minutes	15 minutes
3	Vacuum Cycle	132°C / 270°F	4 minutes	20 minutes
4	Vacuum Cycle	134°C / 273°F	3 minutes	20 minutes

▲ Table 1: Autoclave settings

## Description of the operating procedure



**STEP 1.** (Optional) Attach EasyPrep Mouthpiece to EasyPrep Adaptor. Check to make sure the two parts are correctly locked in place.



**STEP 2.** Gently slide the folded EasyPrep. Mouthpiece into the buccal vestibule on the side of the mouth that is receiving treatment.



**STEP 3.** Seat the bite block in the contralateral side, between the opposing teeth.

## Contact Details

**Australian Sole Dealer :**  
 Company: Preventive Dental Pty Ltd  
 Website: <http://dentalfaceshield.com.au/>  
 Email: [info@dentalfaceshield.com.au](mailto:info@dentalfaceshield.com.au)

**The Principal:**  
 Company: CEREC Asia Co., Ltd.  
 Address: 8F, No 356, Sec 1, Neihu Rd, Neihu Dist,  
 Taipei City, 11493  
 Taiwan

**Manufacturer:**  
 Company: Megaforce Co., Ltd.  
 Address: 1F-2F, No.5, and 1F-2F No.7 Ziqiang St.,  
 Tucheng Dist., New Taipei City, Taiwan R.O.C.

Made in Taiwan