


CHARMANT StealthShield Comb model Instruction for Use EN

General Information

Product Name : StealthShield
 Model Name : Comb model
 General Name : Face Shield
 Intended Use : Wear on the face to protect splash adhesion
 Product Category : Personal Protective Equipment (PPE)
 Size / Color Variation : F(One size) / Brown
 Configuration : Frame 1 pc, Film 1 sheet

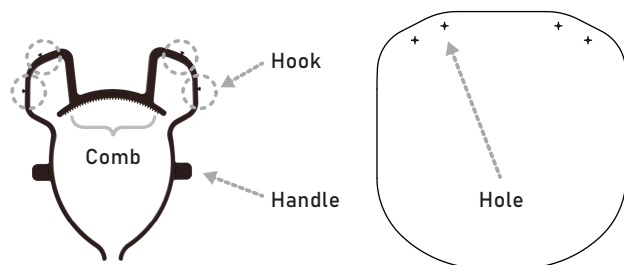

Symbols of labelling

Symbol	Description
	Manufacturer
	Contraindications / Danger / Warning / Precautions
	Non-Sterile
	Read Instruction for Use
	European Authorized Representative
	Lot Number
	Catalog Number
	Complies with requirements of EU Regulation 2016/425

How to Assemble

- 1) There is no distinction between 2 sides of the frame.
- 2) Fit the 4 holes of the film to the hooks of the frame respectively.
- 3) Make sure the hole pressed in properly and edge of the hole is flat.



- 4) Pinch the handles and put it on the head by spreading it out a little.
- 5) Adjust the position.
- 6) When taking off the shield, use the handle; when putting away the shield, gently remove the film from the frame.



Warnings and Precautions

- Don't use the shield if any component is damaged or disable to replace with new component.
- Single use or multiple uses by the same user when disinfected properly.
- This shield is supplied clean but NOT sterilized.

Reprocessing Instruction

In case of reuse, after preceding use, disinfect / sterilize in accordance with guidance provided by the CDC and EPA. (<https://www.epa.gov/coronavirus/about-list-n-disinfectants-coronavirus-covid-19-0>)
 Alcohol-based disinfectant may cause cloudy white.
 Able to wash with detergent, rinse, and wipe off with a soft cloth.
 Before reuse, inspect the frame and the film for damage. (cracks, deformation, impaired vision, or improper fit)

Technical & Safety Info.

Element	Attribution
Material (Frame)	polypropylene
Material (Film)	Polyethylene terephthalate
Gases during combustion	Aldehyde gas
Ignition Temp (Frame)	130°C / 266°F
Ignition Temp (Film)	300°C / 572°F
Disposal Method	Follow Local regulation

USE CASE



Care



Clinic



Nursing



School



Barber



Esthetic



Transport



Delivery



Shop



Restaurant



Reception



Meeting



Manufacturer Info.

CHARMANT INC.

6-1 Kawasari-cho, Sabae-city, Fukui-pref.
 916-8555 JAPAN
 Phone. 0120-180807(JP) / +1-973-867-2450 (US)
 medical@charmant.co.jp www.charmant.com

